

TIA PREMIUM

GOLD FILE

Variable Taper And Off-centered Rectangular Cross Section

Material

New Generation Of Raw Material

Efficiency

Advanced Technical Process Shocked Toughness

Safe

200% ~ 500% Fracture Resistance Improved



www.tiadent.com



| Size | Taper | Length | Torque | Speed | Cross Section | Packing |
|-------|-------|------------|-----------|--------|---------------|----------|
| ● TX1 | 04 | 21,25,31mm | 2-5.0N/cm | 300rpm | | 3pcs/box |
| ● TX2 | 06 | 21,25,31mm | 2-5.0N/cm | 300rpm | | 3pcs/box |
| ● TX3 | 07 | 21,25,31mm | 2-5.0N/cm | 300rpm | | 3pcs/box |
| ● TX4 | 06 | 21,25,31mm | 2-5.0N/cm | 300rpm | | 3pcs/box |
| ● TX5 | 06 | 21,25,31mm | 2-5.0N/cm | 300rpm | | 3pcs/box |



UNIQUE ASYMMETRIC ROTARY MOTION FOR ADDED EFFICIENCY

Tia Premium Gold Files AR Motion allows you to achieve a fully tapered canal with fewer files. The result is the predictable Tia Tornado Gold File shape you expect — with greater procedural efficiency.

SHORTENS THE SHAPING TIME

A shorter clinical sequence means that less time is spent changing instruments. For the dentist, this is valuable time that can be used for other procedures, such as irrigation.

BRINGS IMPROVED SAFETY

The risk of file breakage is significantly decreased, at the same time, respect of the original root canal anatomy is greatly improved. Instrument performance and patient safety are essential to good endodontics.

PROTOCOL FOR USE

- 1- Prepare straight — line access to canal orifice.
 - 2- Use K-files (#10, #15) to detect the root canal.
 - 3- Use (TSX) to enlarge upper part of the canal.
 - 4- Use K (#15) or Tia Pathglider.
 - 5- Use (TX1) to (2/3) of the canal (brush and follow).
 - 6- Use (TX2) to (2/3) of the canal (brush and follow).
 - 7- Use (TX1) to full working length (brush and follow).
 - 8- Use (TX2) to full working length (brush and follow).
 - 9- If the size of K-file (#25) is loose at length, then continue shaping with (Tx3), (Tx4) or (TX5), gauging after each instrument with K-file (#30, 40 or 50).
- *During protocol of use, irrigate, recapitulate after each file.

Tiaendo

TiaDent Inc.

1627 Cortex Midtown Pl Rosenberg, TX 77471 ☎ + 13184800797

